

TruVue Lite™ – Skills Checklist



Purpose Statement:

Verification that staff are trained on the proper use of the TruVue Lite

Performance Criteria	Performs Independently	Needs More Training	Comments
States that the TruVue Lite Heel Protector is indicated for the prevention and management of heel pressure injuries, preventing hip rotation (<i>optional wedges required</i>), and providing foot protection.			
Size: <ul style="list-style-type: none"> TruVue Lite - One size fits most 			
Application of heel protector unassembled: <ul style="list-style-type: none"> Places foot onto heel protector and aligns ankle with top of heel opening Assembles foot gate comfortably around the foot Secures the heel protector with black straps Places hand under the heel to ensure that the heel is offloaded 			
Application of heel protector assembled: <ul style="list-style-type: none"> Places foot into heel protector and secures with black straps Places hand under the heel to ensure that the heel is offloaded 			
Removes heel protector each shift and as needed for skin assessment			
Removes heel protector during chair transfers and/or ambulation			
Uses tubing ports to direct tubing away from patient's skin			
Re-adjusts heel protector to ensure proper placement after patient repositioning			
Uses wedge as needed to help prevent hip and leg rotation			
Cleans with a damp cloth			
States that the product is single patient use			
Sends the product with patient to the next level of care			

Employee Signature: _____

Date: _____

Evaluator Signature: _____

Date: _____

